

Specimen Instructions

FFPE Specimens (Block or Slides)

DO NOT USE strong acids (e.g. hydrochloric acid, sulfuric acid, picric acid) as these destroy nucleic acid. When decalcification is required, brief exposure to a weak acid such as EDTA is recommended.

SAMPLE TYPE

1 FFPE BLOCK OR 16 UNSTAINED SLIDES (+ 1 H&E SLIDE)


Tissue should be formalin-fixed and embedded into a paraffin block. Use standard fixation methods with 10% neutral-buffered formalin. **DO NOT** use other fixatives (AZF, B5, Bouin's, Holland's). If sending slides, send 16 unstained slides (charged and unbaked, with tissue cut at a 5 micron thickness), plus 1 H&E slide.



SURFACE AREA

2 OPTIMUM: 5 x 5 mm²


Tissue should have a surface area of at least 25 mm² (5 x 5 mm², 2.5 x 10 mm²).

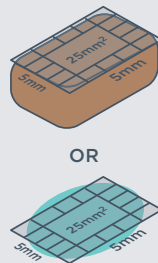

SURFACE VOLUME


3 OPTIMUM: 2 mm³

Optimal sample volume can be achieved by sending optimal tissue surface area (25 mm²) at a depth of ≥80 microns. For suboptimal tissue surface area, additional depth is required.

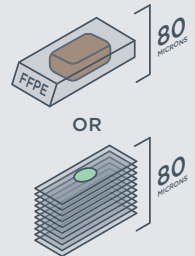

NUCLEATED CELLULARITY


4 DNA is extracted from nucleated cells. Samples with low nucleated cellularity (e.g., those with abundant mature erythrocytes, lesional cells that contain excessive cytoplasm, or tissue with extensive associated fibrosis) may require greater tissue volume to yield sufficient DNA at extraction.

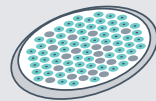

TUMOR CONTENT

5 MINIMUM: ≥20%

If the ratio of nucleated malignant to nucleated non-malignant cells is too low, sensitivity of detection of certain classes of alterations is reduced. High tumor content is preferable. **Note for liver specimens:** Higher tumor content may be required because hepatocyte nuclei have twice the DNA content of other somatic nuclei.

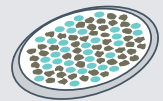

Note: All cytologic and histologic specimens will be reviewed internally by a pathologist, and a determination of sample adequacy will be made. Additional or alternate material may be requested for optimal analysis.

Specimen Instructions

Sarcomas

DO NOT USE strong acids (e.g. hydrochloric acid, sulfuric acid, picric acid) as these destroy nucleic acid. When decalcification is required, brief exposure to a weak acid such as EDTA is recommended.

SAMPLE TYPE

1 FFPE BLOCK OR 16 UNSTAINED SLIDES (+ 1 H&E SLIDE)


Tissue should be formalin-fixed and embedded into a paraffin block. Use standard fixation methods with 10% neutral-buffered formalin. **DO NOT** use other fixatives (AZF, B5, Bouin's, Holland's). If sending slides, send 16 unstained slides (charged and unbaked, with tissue cut at a 5 micron thickness), plus 1 H&E slide.



SURFACE AREA

2 OPTIMUM: 5 x 5 mm²


Tissue should have a surface area of at least 25 mm² (5 x 5 mm², 2.5 x 10 mm²).

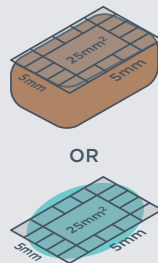

SURFACE VOLUME


3 OPTIMUM: 2 mm³

Optimal sample volume can be achieved by sending optimal tissue surface area (25 mm²) at a depth of ≥80 microns. For suboptimal tissue surface area, additional depth is required.

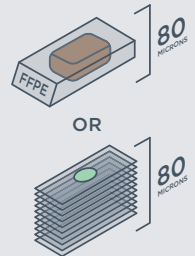

NUCLEATED CELLULARITY


4 DNA is extracted from nucleated cells. Samples with low nucleated cellularity (e.g., tissue with extensive associated fibrosis) may require greater tissue volume to yield sufficient DNA at extraction.

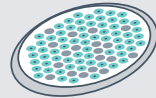

TUMOR CONTENT

5 MINIMUM: ≥20%

If the ratio of nucleated malignant to nucleated non-malignant cells is too low, sensitivity of detection of certain classes of alterations is reduced. High tumor content is preferable. **Note for liver specimens:** Higher tumor content may be required because hepatocyte nuclei have twice the DNA content of other somatic nuclei.

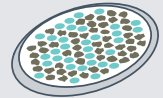

Note: All histologic specimens will be reviewed internally by a pathologist, and a determination of sample adequacy will be made. Additional or alternate material may be requested for optimal analysis.

LOCAL CONTACT

MALAYSIA
Roche (Malaysia) Sdn. Bhd.
Level 21, The Pinnacle, Persiaran Lagoon,
Bandar Sunway, 47500 Subang Jaya,
Selangor Darul Ehsan
Customer Services team: 1800 806 857

For more information, contact
our APAC Customer Services team:
APAC.foundation@roche.com